

# Safe Mitten III

## Mitten-style gloves featuring a dual-layer padded construction

Features a "Idea Hook" that is difficult for the wearer to open.

Breathable mesh fabric on the back of the hand.

With approximately 4.7cm of thick padding, it reduces risk of accidental gripping.



**Ambidextrous**



Stopper with an "Idea Hook" that is difficult for the wearer to open. Attached to the glove with hook-and-loop fastener. Replaceable if needed.

### Replacement Parts

Replacement Stopper with "Idea Hook"



Blind Stopper




"Idea Hook" is hidden from view on the outside.

This product is used temporarily to support medical and caregiving procedures when there is a risk of the patient engaging in dangerous behaviors, such as self-harm or pulling out catheters, or when they try to tamper with their diaper.

### "Idea Hook"



**[How to unhook]**  
Support the back of the hook with your finger, and press the center with your fingernail.

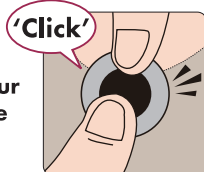

**[How to close]**  
Place both hands on the edges of the hook over the flap and press firmly until you hear a 'click'.

### Size

Wrist	13 ~20cm
Hand Length	14 ~19cm



### Material

Glove fabric : Polyester, Cotton  
Mesh fabric : Polyester  
Padding material : Polyethylene, Polyester  
Strap : Nylon  
Hoop-and-loop fastener : Nylon  
"Idea Hook" : Stainless steel

### Washing Instructions

- Make sure hooks and buckles are closed before washing. Washing them open or detached can lead to wear or damage.



Use a pressing cloth

### Specification

Product Code	Item	Unit
041056	Safe Mitten III	1 Pair
041057	Replacement Stopper with "Idea Hook"	1 Piece
041058	Blind Stopper	1 Piece