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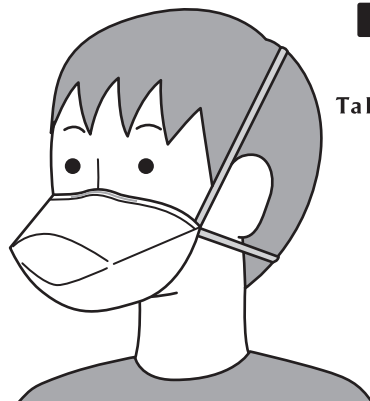
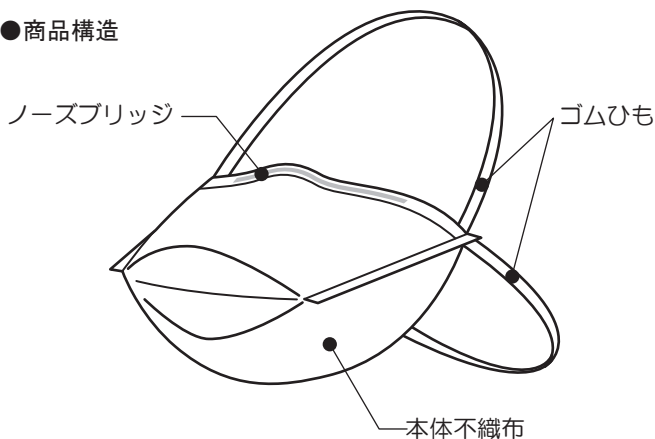
Bend the nosepiece to conform
to your nose.

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With your index fingers and thumbs,
separate the two headbands.

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Hang the lower part on
your chin and bring both bands
over your head.

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Place the upper band at your crown
and the lower band at your neck.

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Mould nosepiece tightly
to ensure a secure facial fit.

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Using both hands, over the whole of the respirator
to check the fit-breathing out to make sure that
no breath escapes from around the nose, cheeks or
edges of the respirator.

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FILTERING FACEPIECE APPROVAL LABEL

TAKETORA Co., Ltd.
9279-69, Oroshihoncho, Seya-ku,
Yokohama-shi, Kanagawa 246-0001, Japan
TEL: +81-45-921-3552

This respirator is approved only in the following configuration:			
TC-	Protection ¹	Respirator	Cautions and Limitations ²
84A-3348	N95	076073	ABCJMNOPS

1. PROTECTION:

N95-Particulate Filter (95% filter efficiency level)
effective against particulate aerosols free of oil;
time use restrictions may apply.

2. CAUTIONS AND LIMITATIONS:

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to User's Instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.
- S - Special or critical User's Instruction and/or specific use limitations apply. Refer to the User's Instructions before donning.

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USER'S INSTRUCTIONS

WARNING!

Although this respirator is useful from the particulate substance with detrimental nature to protection, the risk concerning sickness or infection is completely non-avoidable. Failure to read and follow all warnings and instruction will reduce respirator performance and may result in sickness and / or death.

[USE INSTRUCTION]

This respirator does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.

Use for protection against particulate aerosols free of oil.

Do not use for gasses, vapors asbestos, paint spray or any oil-based aerosols, or in environments.

Before occupational use of this respirator, a written respiratory protection program must be implemented meeting all the requirements of OSHA 29 CFR 1910.134 such as training, fit testing and medical evaluation.

Leave the use environment immediately in the case of dizziness, nausea, breathing difficulty or other symptoms of physical distress.

Don a new respirator if a proper fit cannot be obtained, if breathing becomes difficult, or the respirator becomes damaged.

Only for single use.

Store unused respirators in a clean, cool, dark, dry place.

The model, 076073 filtering facepiece respirator has been manufactured by Approval Holder, San-M Package Co., Ltd. for TAKETORA Co., Ltd. under TC-84A-3348.

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